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VISAID ALOE MONODOSIS is a lubricating, isotonic and preservative-free ophthalmic solution with hypromellose, aloe vera and essential electrolytes, for ocular administration.

COMPOSITION:

Hypromellose 0.2 g, Aloe vera, Sodium chloride, Magnesium chloride, Potassium chloride, Calcium chloride, Boric acid, Borax decahydrate, Purified water 100 ml

NO PRESERVATIVES

USAGE:

VISAID ALOE MONODOSIS is intended for:

Relief of dryness and eye irritation. These symptoms can arise due to various factors, such as pollution, dry or dusty environments, smoke, prolonged use of contact lenses, chlorinated water (swimming pools) or other irritants.

INSTRUCTIONS FOR USE:

Before using the product, wash your hands with soap and water.
Take a vial of VISAID ALOE MONODOSIS and open it by turning the cap.
Apply one or two drops into each eye and blink several times.
VISAID ALOE MONODOSIS can be used up to 8 times a day.

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Date of last revision: 2015/10

PRECAUTIONS:

Do not ingest; keep out of reach of children.
Each vial is for one use only.
Once the vial is opened, use immediately. This product does not contain preservatives and must be discarded after use to prevent microbial contamination.
Do not use if the blister is damaged.
Store at room temperature.
Do not use after the expiry date.

INTERACTIONS:

Do not use in conjunction with any medication or other product that is applied to the surface of the eye, to prevent any alteration of the effects of both products.

CONTRAINDICATIONS:

Do not use the product if you are allergic to any of its components.

ADVERSE EFFECTS:

In case of eye irritation or adverse reaction, please consult a specialist.